

FAT BURNING KITS

FACE & BODY

VIANACARE.COM



Table of contents

- 1. General description of the set**
- 2. Indications**
- 3. Advantages and key features of the set**
- 4. VK025 application protocol (Hyaluron Pen)**
- 5. Ways of treatment. Dosages.**
- 6. Required accessories and disposable items**
- 7. Counter-indications**

• GENERAL DESCRIPTION OF THE SET

It's worth noting that set consists of the VK025 next generation lipolytic agent and a specially created needle-less Hyaluron Pen Black injector intended to administer the product without injections.

The VK025 lipolytic agent allows eliminating fat deposits and contains only natural ingredients*: sodium chloride, marroni extract, pulsatilla extract, walnut extract, crimson flowers, fucus.



⚠️ Despite the fact that the lipolytic agent consists of natural ingredients, one needs to follow the rules of the agent dilution in order to avoid side-effects and complications (see section 4).

After reaching fat deposits, the lipolytic agent stimulates the process of fat decomposition, thus creating fatty acids. In order to avoid the body to use the fatty acids to create new deposits of fat, the lipolytic agent contains several important ingredients, which increase the speed of fatty acids removal from the body by using the lymphatic system.

Kindly note that this procedure will work only in case if you are able to reach the required location of fat deposits, i.e. buccal fat pads.

FAT BURNING SET FOR FACE contains VK025 lipolytic agent (10 ampoules by 2ml).

FAT BURNING SET FOR BODY contains VK025 lipolytic agent (5 ampoules by 8ml).

2. INDICATIONS

Indications


Fat deposits on the face and the body, lower third ptosis, prolapses.

Zones of lipolytic agent administration

- neck
- chin
- body
- arms
- legs
- nose tip
- cheeks

⚠ Dangerous zone (prohibited)

Periorbital

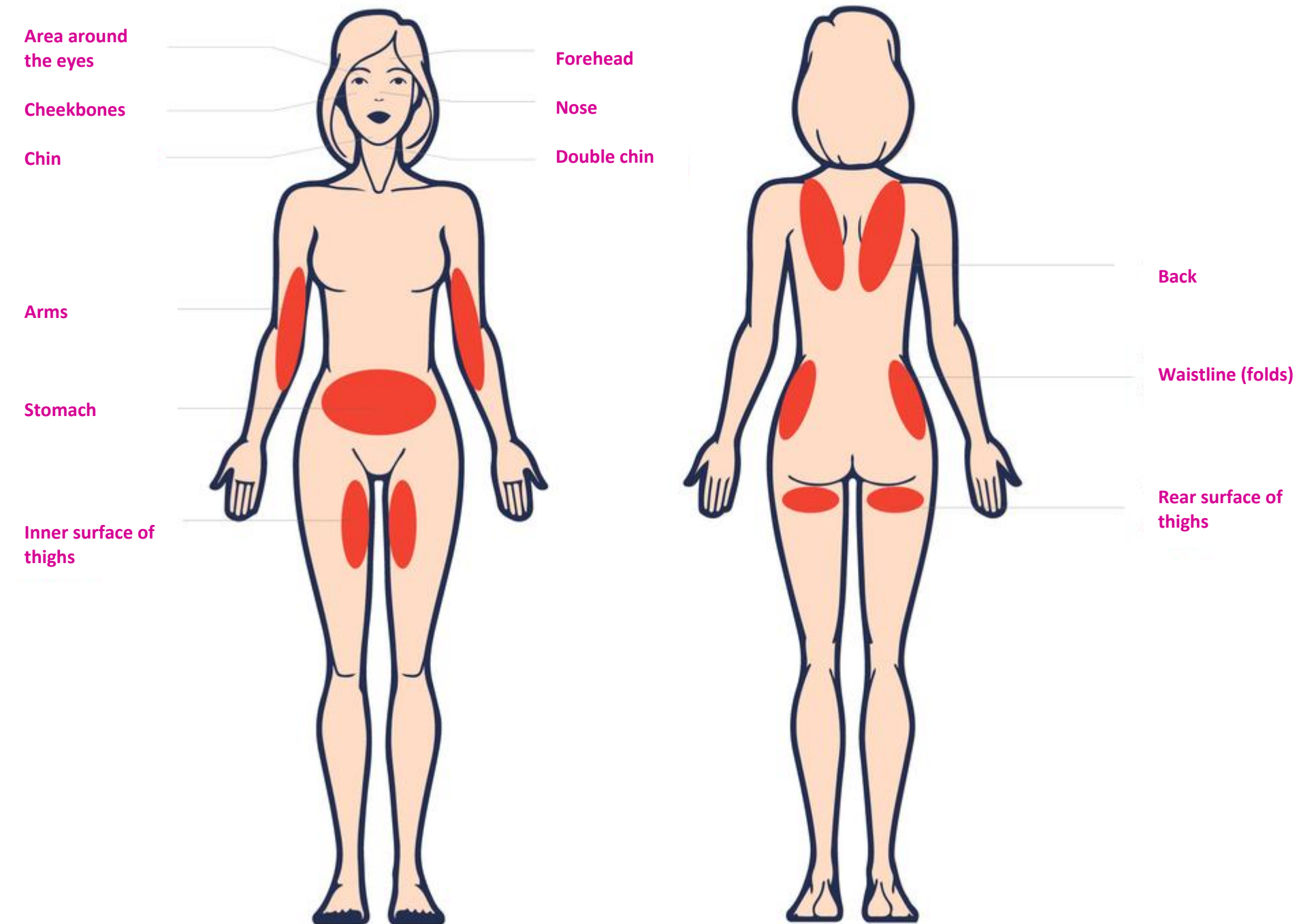

3. ADVANTAGES OF THE SET


VK lipolytic agent is one of the most modern, innovative and popular lipolytic agents in the market of cosmetological products.

The key **advantage of the Hyaluron Pen usage with the lipolytic agents is the following: one is able to administer the lipolytic agent in the most correct way**, while **keeping the proper depth of administration**, adjusting the dosage of the lipolytic agent, and reaching the deepest layers of the skin **without damaging the skin structure**. The key target is the fat pad in this case.

Furthermore, this way of treatment allows eliminating all side-effects and complications, as well as to avoid any rehabilitation period. By using the needle-less injector in a right way, one is able **to avoid bruises, swelling and hematomas**.

The Hyaluron Pen injector allows delivering various agents to the required depth. The higher is the dosage of the product, the higher is the administration depth.

Administration of the lipolytic agent by using the Hyaluron Pen injector

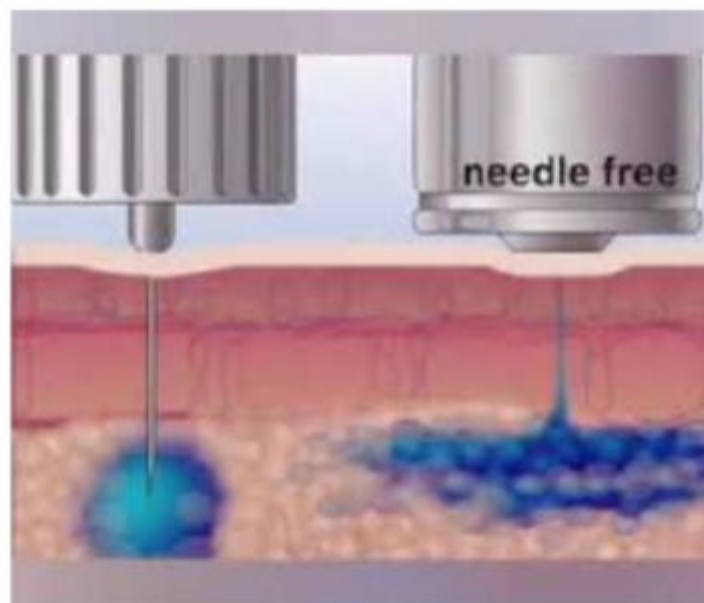

After reaching the fat pad under a high pressure, the lipolytic agent will be administered under the skin in an even matter in order to cover an area of 2x2cm (which is simply impossible in case of a regular needle).


4. VK025 APPLICATION PROTOCOL (HYALURON PEN)

1. Choose an area.
2. Use a white pencil to draw a grid on the skin (each section of the grid has to have the following dimensions: 2x2cm).
Draw a dot in the middle of each section in order to administer the product. At the end of the procedure one needs to remove the grid from the skin.
3. Degrease the skin surface (one can use any antibacterial agent).
4. Take a disposable syringe and use the needle adapter to fill the syringe with the required amount of diluted lipolytic agent. Kindly note that the maximum amount of pure substance is 5ml and 20ml for the face and the body respectively.
5. Install the syringe into the Hyaluron Pen. In order to make a single 'shot', one needs to choose the amount of the agent by twisting the device. The syringe has 5 sections. The overall volume of the syringe is 0.5ml, thus one section contains 0.1ml
6. Use the 'pinch' method to perform the treatment.
7. Remove the grid from the skin and massage the area of treatment. Kindly note that the massage session is not obligatory because the product was already administered under the skin in an even matter thanks to the usage of the needleless injector.
8. Your customer can go home now.

Required number of sessions: 4-5 sessions every 14-20 days

Kindly note that the administration method depends on the chosen area

Area of administration	Lipolytic agent	Sodium chloride	Max. daily dosage
Face (1:2)	1 ml	2 ml	5 ml
Body (1:1)	1 ml	1 ml	20 ml

 **Strictly follow the dilution rules!**
In order to make it, one needs a 2ml syringe and normal saline

5. WAYS OF TREATMENT. DOSAGES

5.1. How to use on the face How to dilute the lipolytic agent with 1:2 ratio



The maximum amount of the pure product for the face is the following: 5ml.

RED CIRCLE

2 sections (0.2ml) for each 'shot'
at the distance of 2cm

YELLOW CIRCLE (Delicate zones)


1.5 sections (0.15ml) for each 'shot'
at the distance of 1.5cm

! WARNING!

The higher is the amount of the agent in one 'shot', the deeper is the administration of the product under the skin.

5. WAYS OF TREATMENT. DOSAGES

5.2. How to use on the body How to dilute the lipolytic agent with 1:1 ratio

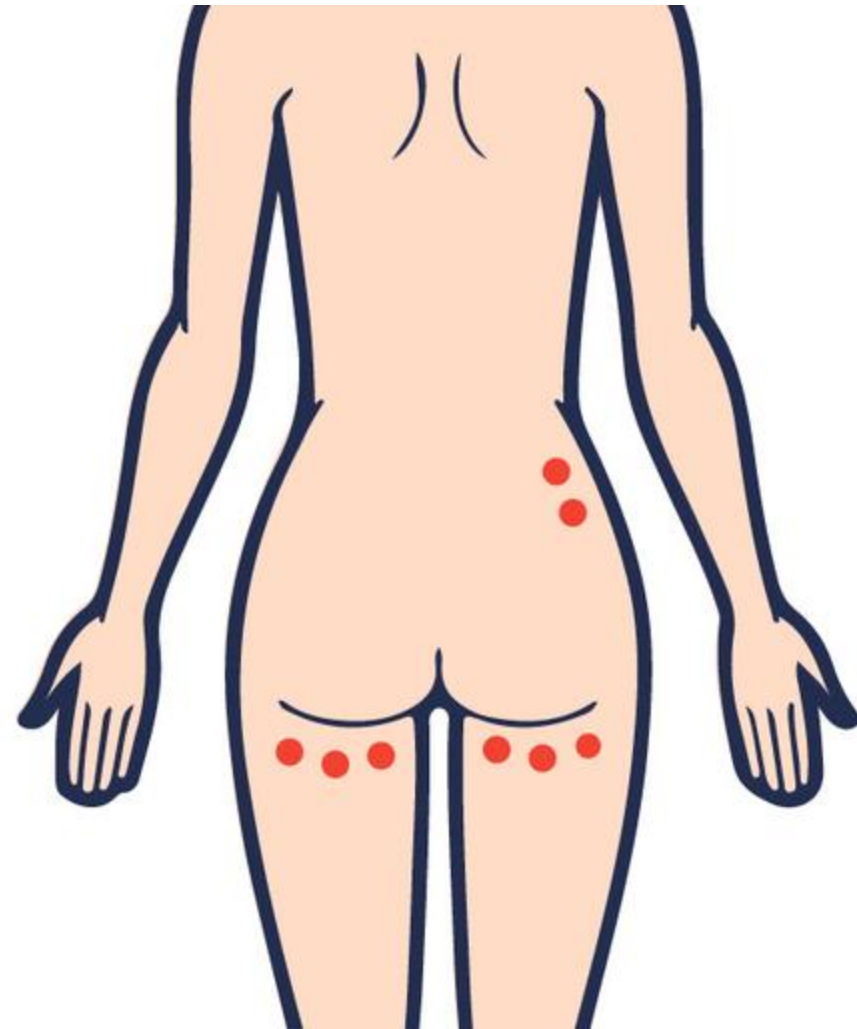

The maximum amount of the pure product for the face is the following: 20ml.

RED CIRCLE

3 sections (0.3ml) for each 'shot'


One needs to make sure that each shot is made into a separate area with the dimensions of 2x2cm.

! WARNING!

The higher is the amount of the agent in one 'shot', the deeper is the administration of the product under the skin.

5. WAYS OF TREATMENT. DOSAGES

5.3. How to use in the periorbital zone

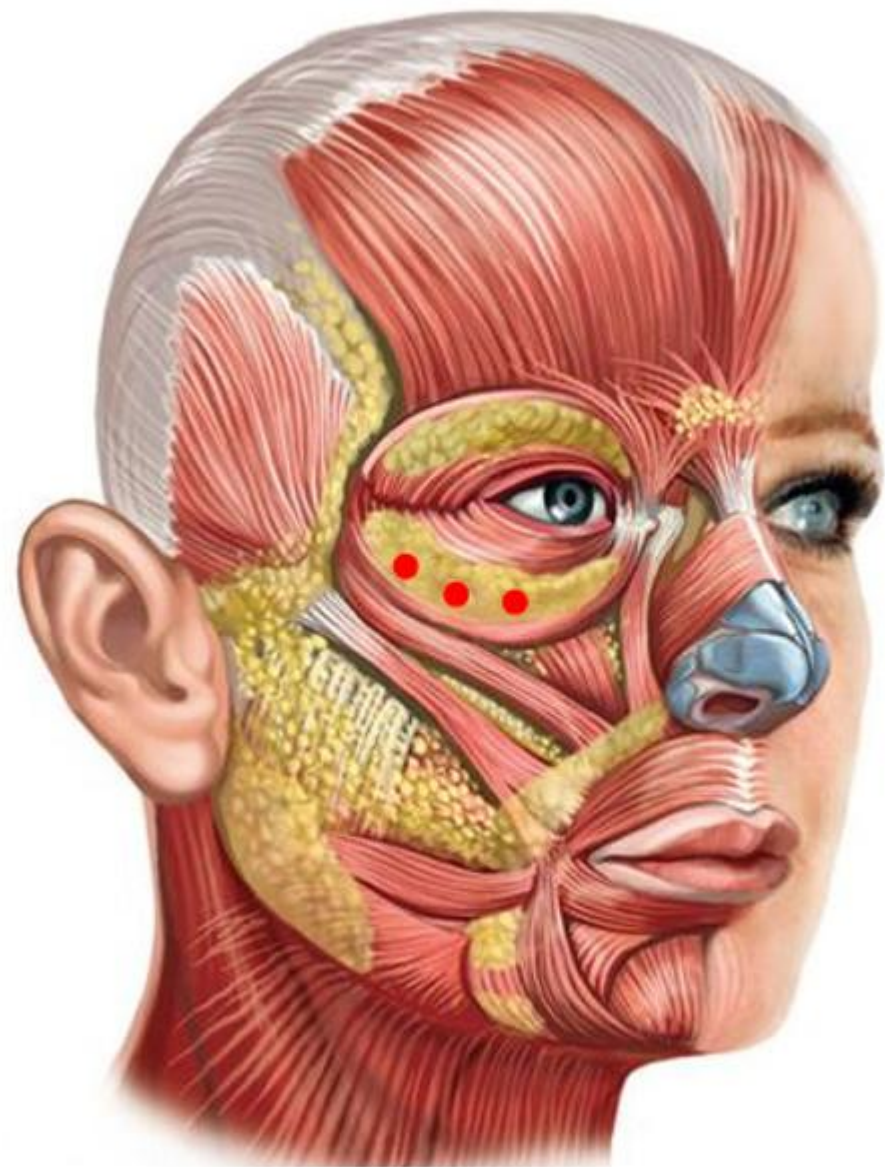

Hold the fat pad with your fingers and make 'shots' at the distance of 1.5cm without adjusting the required volume of the active substance.

! WARNING!

- **Kindly note that only advanced users of the Hyaluron Pen can use this way of treatment**
- **One needs to follow the key principles of the 'pinch' method in order to avoid hematomas and bruises**
- **If you avoid twisting the Hyaluron Pen in order to choose the required amount of the product, the entire syringe volume will be administered.**

5. EXPENDABLE MATERIALS

- Antibacterial solution
- Cotton pads
- White pencil
- Lipolytic agent
- Sodium chloride
- Syringe adapter
- Attachment for the Hyaluron Pen
- Disposable 2ml syringe
- Antishock first-aid kit





6. COUNTER-INDICATIONS FOR TREATMENT

- Clinical autoimmunity
- Pregnancy or breastfeeding
- Varicosity with trophic skin changes
- Dermatological diseases in the area of administration
- Diseases of kidneys, liver and pancreatic gland
- Idiosyncrasy
- Collagen diseases
- Intake of blood-thinners, which affect the blood clotting ability
- Severe cardiovascular diseases
- Chronic infections
- Endocrine disorders, i.e. diabetes